Instructions for Use

Boston Spinal Brace

1. Intended use

Boston Scoliosis Brace	Boston Soft Body Jacket Boston Lite	Boston Overlap Brace	Boston Body Jacket
A prefabricated modular spinal orthosis used in the non-operative and conservative treatment of Idiopathic Scoliosis primary used for adolescent and young females	An orthosis to aid posture control and support in the orthotic treatment of various neuromuscular associated conditions primarily used by adolescent and young adult population with respect to indications outlined below.	A prefabricated LSO Spinal Brace used to aid in the orthotic treatment of chronic or temporary low back pain primarily used by adults with respect to indications outlined below	A prefabricated spinal orthosis used to aid in the orthotic treatment associated with various conditions requiring immobilization of the spine used by paediatric and adults with respect to indications outlined below.

2. Indication / Contraindication

Boston Scoliosis Brace	Boston Soft Body Jacket Boston Lite	Boston Overlap Brace	Boston Body Jacket
Indications:	Indications:	Indications:	Indications:
Adolescent Idiopathic Scoliosis (AIS)	Neuromuscular Scoliosis Cerebral Palsy Myelomeningocele Muscular Dystrophy Postural Positioning Improve Seating Prospective Protection Pain Relief	Post Operative Stabilisation Compression Fracture Spondylolisthesis Discogenic Low Back Pain Muscular Dystrophy Osteoporosis	Post Operative Stabilisation Compression Fracture
Contraindications:	Contraindications:	Contraindications:	Contraindications:
Thoracic Lordosis Severe obesity	None known	None known	None known

General Safety Instruction

- This orthosis is for single patient multiple use
- Only appropriately trained and qualified Clinicians should fit your orthosis with specific reference to the Technical Manual available to view and download at www.allarduk.co.uk
- Patients should contact their Clinician in event of any persistent skin irritation
- In the event of incorrect use product liability is excluded

Application Information

Your Clinicians instruction will always supersede the information given in this instruction!

You have been prescribed this orthosis to treat a particular condition. Treatment can vary for each patient. Your clinician will provide specific information about:

- Applying your orthosis
- How to get used to wearing your orthosis to achieve optimum hours of daily wearing.
- Duration of use
- Acceptable and unacceptable activities

Side Effects

The skin under the orthosis should be checked regularly for irritation. Due the normal pressure of the orthosis on your skin while wearing for extended lengths of time your skin will need to adjust. We recommend that you:

- shower or bathe daily;
- apply / massage rubbing alcohol to your skin where the brace touches your body. This may help to toughen the skin. Pay attention to where the skin has become pink (where the most pressure is);
- wear a seamless, tight-fitting vest under the orthosis which covers the body under the orthosis. Contact your clinician if any irritation that occurs does not heal.

Maintenance and Cleaning

It is important to clean the orthosis daily. Use a damp cloth and soapy water to wipe it clean. Dry thoroughly.

CE Conformity

The product satisfies the requirement regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears a CE mark.

Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Warranty and Guarantee

We recommend that the orthosis is renewed / changed if it no longer fits or is damaged.

Warranty is provided under the terms of sale and supply of Allard Support UK Ltd provided that the above conditions are met.

Storage and Disposal

When not in use store your orthosis safely to avoid accidental damage. Refer to your local disposal regulations with reference to the material content as per the material content symbol on your orthosis

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2. Indication / Contraindication

Boston Scoliosis Brace	Boston Soft Body Jacket Boston Lite	Boston Overlap Brace	Boston Body Jacket
Indikasjoner:	Indikasjoner:	Indikasjoner:	Indikasjoner:
Adolescent Idiopathic Scoliosis (AIS)	Neuromuscular Scoliosis Cerebral Palsy Myelomeningocele Muscular Dystrophy Postural Positioning Improve Seating Prospective Protection Pain Relief	Post Operative Stabilisation Compression Fracture Spondylolisthesis Discogenic Low Back Pain Muscular Dystrophy Osteoporosis	Post Operative Stabilisation Compression Fracture
Kontraindikasjoner:	Kontraindikasjoner:	Kontraindikasjoner:	Kontraindikasjoner:
Thoracic Lordosis Severe obesity	None known	None known	None known

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