allard^u_k

BOSTON HIP ABDUCTION BRACE

PATIENTINFO / PASIENTINFO / POTILASINFO
PATIENTINFO / PATIENT INFO / PAZIENTEINFO
PATIENTINFO / PATIËNTINFO

SVENSK • NORSK • DANSK • SUOMI • ENGLISH
FRANÇOIS • ITALIANO • DEUTSCH • NEDERLANDS

HÖFTABDUKTIONSORTOS

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HIP ABDUCTION

-Wear and Care Guide

Your doctor/orthotist has prescribed this brace to assist with a particular condition. As the treatment can vary from case to case your doctor or orthotist will provide specific information about:

- Applying your Orthosis
- Weaning (Acclimatization) schedule to achieve optimum hours of daily wearing (if applicable).
- Duration of use
- Acceptable activities
- Activities to avoid
- Who to contact in case of problem

Cleaning your Orthosis:

- It is important to clean the orthosis on a daily basis. For best results use rubbing alcohol. Alternatively use a soapy cloth and rinse with a damp cloth and towel dry.
- Always check your orthosis before putting it back on to ensure that it is undamaged and free from dirt.

Skin Care:

It is very important to prevent skin breakdown (sore, red, raw skin). The skin underneath the orthosis needs to be toughened up, especially where the orthosis applies the most pressure.

- Pay special attention to pink areas of the skin where orthosis pressure is highest.
- For children wearing a diaper (nappy), the brace is fitted over the diaper (nappy).
- This brace can be worn over clothing such as tights, track suit bottoms, leggings, etc.
- If skin irritation/breakdown persists, contact your fitter or clinic for advice.

NOTE:

Any special Doctor's / Orthotist orders should always supersede the general guideline information given in this guide. This product is to be fitted by an appropriately trained and qualified professional and with reference to the specific product Technical Sheet, available to view and download at www.allarduk.co.uk

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©AllardUK, april 2021
Allard Support UK, Ltd.