**35209**  
**SELECTION®**  
**MULTIOPEN**

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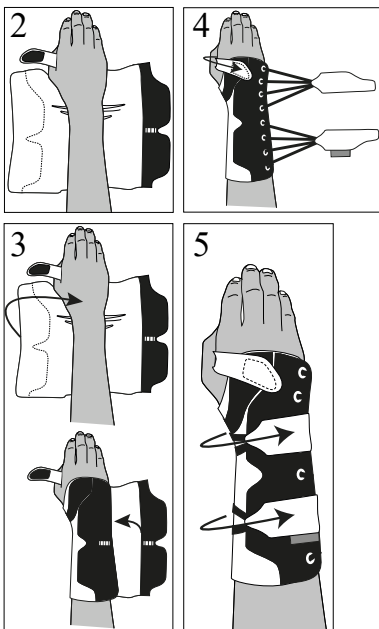
**Intended use**

Intended to off-load and support the wrist to a neutral resting position to maintain important wrist functions.  
For one persons use only.

**Indications:**

Post healed distal radius fracture, Rheumatic disorders, Soft tissue injuries, work related strain injuries or with degenerative changes.

**Contraindications**

Not to be used for non-healed fractures.

**Adjustments**

This product should only be altered by medical professional person.

**Warnings**

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Do not use product when wet or on wet skin. Check your skin on a daily basis.

**Material Specifications**

This product is made from polyester, nylon and elasthan. Spoon is made from aluminum. This product is not flame retardant. Latex free.

**Washing Instruction**

Close all Velcro prior to laundering. Remove spoons before laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

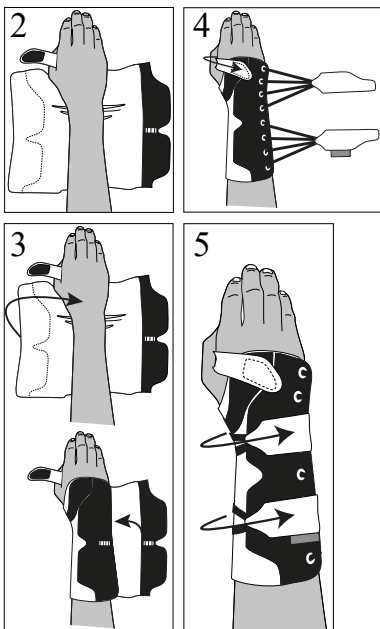
**Who to Contact**

If any problems occur, remove the support and contact your fitter or store where purchased.

## Instructions for use

1. Check that the support is complete and without damage.
2. Open the lacing and Velcro straps. Position the palm so that the stay rests comfortably on the inside.
3. Gently fold over the Velcro part so that the orthosis fits loosely around the arm.
4. Close the thumb strap over the dorsal side of the hand.
5. Then tighten the lacing.
6. Adjust for comfort.

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