35207

**SELECTION  
OPEN WRIST**

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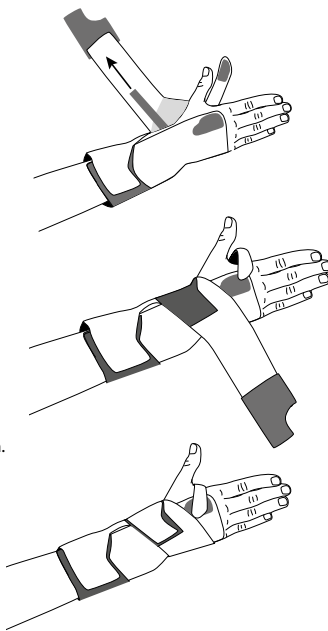
**Intended use**

Selection Open Wrist is intended to be used when the wrist needs off-loading in conditions such as rheumatic disorders or after healed fractures.

**Indications:**

Inflammations, work related strain injuries or when rheumatic and/or degenerative changes are present.

**Contraindications**

No known.

**Adjustments**

This product should only be altered by medical professional person.

**Warnings**

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

**Material Specifications**

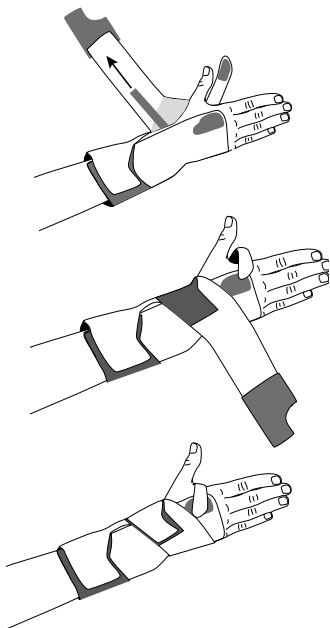
This product is made from polyester, nylon and elastan. Spoons are made from plastic. This product is not flame retardant.

**Washing Instruction**

Close all Velcro prior to laundering. Remove spoons before laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

**Who to Contact**

If any problems occur, remove the support and contact your fitter or store where purchased.

**Instructions for use**

1. Check that the support is complete and without damage.
2. Open all Velcro straps and position the support with the splint inside the hand.
3. Secure the short upper Velcro strap and the thumb strap.
4. Then pull the long strap and wrap it around the wrist.
5. Adjust the short upper strap.
6. Adjust for comfort.

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