**35701
Selection
Minor WRAP**

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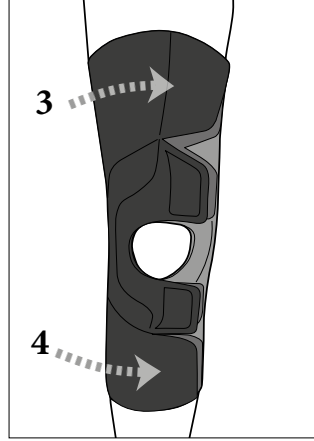
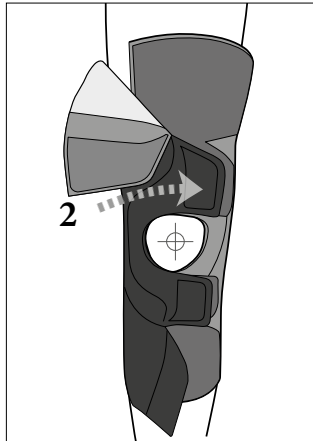
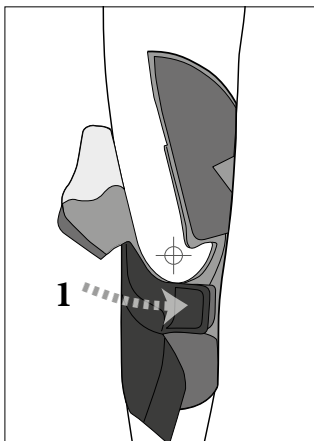
Intended use

This product is intended for treatment of mild conditions such as pain and inflammation. The support gives a light compression and keeps the knee warm. This support is only intended for one person use.

Indications

Knee pain in weight bearing, minor injuries such as stress in ligaments or mild sprains.

Contraindications

Severe knee injuries such as ACL/PCL ruptures. Be careful: Severe swelling problems and blood circulation problems.

Adjustments

This product should only be altered by a medical professional person.

How to use the support

Check that you have the correct size. Some conditions can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gra-

dually. The support should not be used overnight.

Warnings

Do not use directly on open wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from: Polyamid, polyurethane and spandex with metal stays. This product does not contain Latex, and is not flame retardant.

Washing Instruction

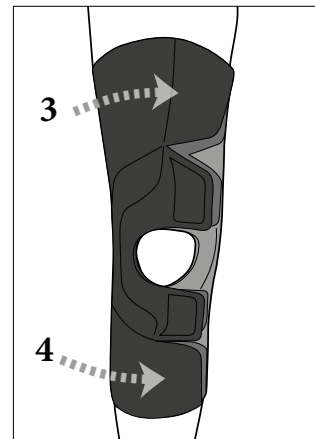
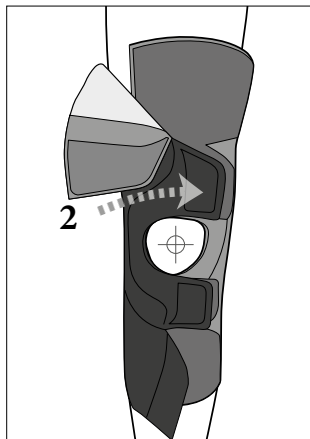
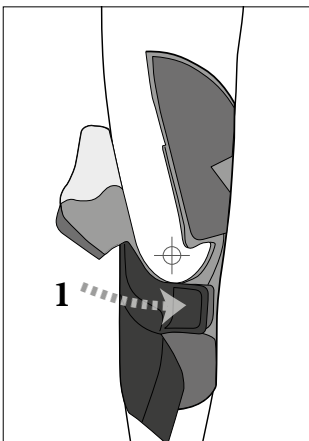
Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

INSTRUCTIONS FOR USE

1. Check that the brace is complete and without damage.
2. Open all the velcro closures so that the brace is flat.
3. Place the open brace behind the knee and start with closing the lower tab under the patella (1). Make sure that patella is centered in the opening.
4. Then close the tab above the knee (2).
5. Then close the other two tabs (3 och 4).
6. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38