allardly

BOSTON
SCOLIOSIS
NIGHT BRACE

PRO

SVENSK
DANSK
NORSK
SUOMI
ENGLISH

FESSIONAL

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

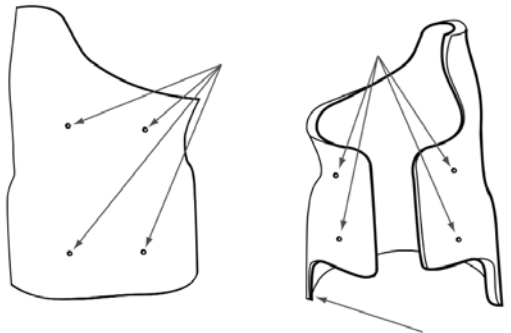
Instruction for fabrication of Boston Scoliosis Night Brace

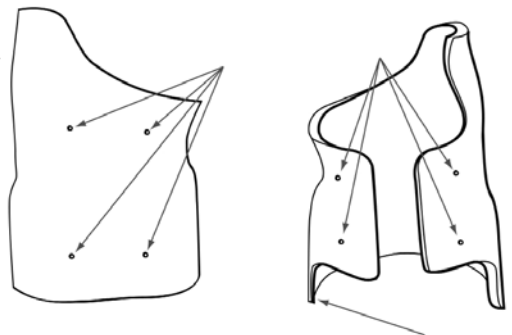1. Separate the outer plastic shell from the internal padding.
2. Begin by cutting down the plastic shell to the trim lines marked on the plastic. Use appropriate tools like a jigsaw.
3. When you have trimmed the plastic shell, use an edge trimmer to remove any burrs and then polish the edges with a soft polishing cone.
4. Insert the plastic shell against the padding. Depending on the orthosis height in the front, two or three straps can be used for the closure. If there is room for three straps, attach them so that they have a different direction on the tension. Mark the location of the straps and drill holes in the appropriate size.
5. When the straps are mounted, merge the two parts (plastic shell and padding). Drill at least nine 6 mm holes (use the plastic sleeves and screws). Be careful not to drill the holes where the padding has increased thickness (pelotts). Merge the two parts. See below for approximate location of the holes
6. When the plastic shell and padding are joined together and the screws are in place, the padding should be trimmed about 2.5 cm above the plastic shell.
7. Sand the edges and then polish with a soft polishing cone.
8. Finally adjust the tongue . Let the tongue be slightly longer than the front piece height and adjust it to the appropriate width. Attach it to the right side of the padding with the enclosed plastic pop-rivets.
9. The orthosis is now ready for fitting.

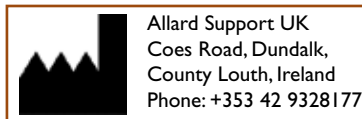Distributed in Scandinavia by:
Camp Scandinavia AB
Karbingatan 38, 254 67 Helsingborg, Sweden
Phone: 46 42 25 27 01

www.allarduk.co.uk • www.camp.se